

Steps for Safe Redonning (Reuse) of N95 Respirators

**N95 may be reused until crushed, soiled, wet
or difficult to breathe through**



1. Wash hands



2. Or use alcohol rub if
hands are not soiled



3. Take out new
mask



4. Put on mask and
fit check



5. Get paper bag, write
your name on bag leave
on counter, and enter
patient room



6. After exiting patient
room remove N95




7. Insert N95 for
later reuse



8. Hands are
contaminated, perform
hand hygiene



9. Upon return
remove 95 from bag,
being careful not to
touch inside of
mask.

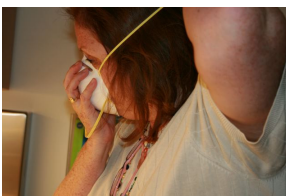

10. Redon N95,
perform fit check



11. Throw away bag
after one use



12. Hands are
contaminated, perform
Hand Hygiene

Never use a N95 respirator for longer than one shift!
Employees are not required to reuse respirators.
Staff should use the N95 respirators for which they were fit tested.

EXAMPLE ONLY